# **Alkaline Detergent** Super Concentrated

## Detergent for Automatic Instrument Washers

#### **Product Features**

### No Neutralizer Required

Safe alkaline single product - no neutralizer required for automatic washers. Excellent as a pre-soak - undiluted product removes heavy stains from stainless steel instruments. Cleans and Protects - provides powerful cleaning action and passivation layer protection for stainless steel surgical instruments.

Counteracts hard water - strong chelating additives help to protect instruments by facilitating complete rinsing, thus denying undesirable hard water minerals the opportunity to form scale. Versatile - formulated for all brands of automatic washer units.

Economical - high level of concentration is cost effective.

#### Directions for Use

**CST-110SCLF** can be used in all types of automatic instrument washing equipment. For high impingement washers: use 1/10-1 fl.oz. per gallon of water. For low impingement washers: can be used in either the wash or neutralizer cycle. For enzyme use, employ CST-404SCLF Enzymatic Super Conc. Cleaner at 0.5 fl.oz. ounce per gallon in the first cycle and CST-110SCLF at 1/10 - 1 fl.oz. per gallon of water in the second cycle. On occasion, these quantities may require adjustment contingent upon water quality and bio-burden load. For heavily soiled instruments, wash cycle time may be increased. Presoak: use CST-404SCLF undiluted as a pre-soak agent for badly stained stainless steel surgical instruments.

#### **Specifications**

Chemical Description:	Concentrated liquid blend of surfactants and chelating agents
Physical Appearance:	Clear, reddish orange, low viscosity liquid for easy metering
pH (1%w/w sol)	10.9-11.6
Specific Gravity	1.14

#### Order Information

CST-110SCLF-1 -	1 gallon plastic bottle, packed 4 per case
CST-110SCLF-2.5 -	$2\frac{1}{2}$ gallon plastic bottle, packed 2 per case
CST-110SCLF-5 -	5 gallon plastic jug with handle
CST-110SCLF-55 -	55 gallon plastic drum

#### Cautions/Warnings

**CST-110SCLF** is a highly concentrated alkaline product. This product should be handled as a corrosive material, which can cause burning of the skin and eyes upon contact. Read material safety data sheet before using this this product. As a safety precaution, protect skin and eyes by wearing NIOSH approved safety goggles and gloves. KEEP OUT OF REACH OF CHILDREN! Refer to MSDS for additional safety information.





Complete Solutions Technologies, LLC 302 N. Washington St., Suite 202 E Moorestown, NJ 08057

www.cstllc.com Toll-Free 888-987-1987 Tel 856-234-7392 Fax 856-234-7918