

Lubricant

SUPER CONCENTRATE

FOR USE IN AUTOMATIC WASHERS

CST-300SCLF - when used in the lubrication cycle of automatic washer/decontaminators, helps protect surgical instruments from corrosion and staining associated with the harsh elements of steam sterilization.

Product Features

Gentle Formula - formulated with gentle, non-aggressive ingredients.

Extends Instrument Life - leaves a protective microfilm which prolongs instrument life.

Frees Up Box Locks - lubricates moving parts of surgical instruments.

Economical - very highly concentrated.

Directions for use

Program machine to deliver **1/20 fluid ounce per gallon rinse water**.

Technical Data

Chemical Description	Water emulsible, non-ionic blend of corrosion inhibitors and gentle lubricants
Physical Appearance	Clear, light yellow liquid with a slight scent
pH	6.8 - 7.5 at recommended dilution ratio
Specific Gravity	0.89

Order Information

- CST-300SCLF-2.5** - 2 x 2.5 gallon plastic jugs per box
- CST-300SCLF-5** - 5 gallon plastic jug
- CST-300SCLF-15** - 15 gallon plastic drum

Caution/Warnings

Care should be taken to minimize exposure. As a safety precaution, wear NIOSH approved safety gloves and goggles.

Refer to MSDS for additional safety information.



COMPLETE SOLUTIONS TECHNOLOGIES
CST

Complete Solutions Technologies, LLC
302 N. Washington St., Suite 202 E
Moorestown, NJ 08057

www.cstllc.com
1-888-987-1987
Tel 856-234-7392
Fax 856-234-7918