

CST-301

INSTRUMENT LUBRICANT

Lubricant for Manual and Machine Use

Product Features

Helps prevent corrosion – anti-corrosion agents inhibit corrosive activity.

Gentle formula – formulated with mild, non-aggressive components.

Extends instrument life – protective micro-film keeps instruments working longer.

Frees up box locks – lubricates moving parts of surgical instruments.

Economical - for manual applications only one part concentrate required for six parts of lubricant bath and ¼ oz. per gallon for automatic washer units.

Directions for Use

For manual use: shake container before each use. In a tray, dilute one part concentrate to six parts water. Mix well. Soak instruments for 30 seconds, remove and let instruments drain.

For machine use: program machine to deliver ¼ fl. oz. per gallon. Proceed with steam sterilization.

Order Information

CST-301-1 – 1 gallon plastic bottle packed four per case (including 1 pump dispenser)

CST-301-5 – 5 gallon HDPE plastic jug with handle

CST-301-15 – 15 gallon plastic drum

CST-301-30 – 30 gallon plastic drum

CST-301-55 – 55 gallon plastic drum

Cautions/Warnings

Avoid prolonged contact with skin. May irritate eyes. Wear eye protection as a safety precaution. Do not swallow. **KEEP OUT OF REACH OF CHILDREN!**
Discard used solution - do not reuse.



Instrument Lubricant Milk



COMPLETE SOLUTIONS TECHNOLOGIES
CST

Complete Solutions Technologies, LLC
302 N. Washington St., Suite 202 E
Moorestown, NJ 08057

www.cstllc.com
Toll-Free 888-987-1987
Tel 856-234-7392
Fax 856-234-7918