

Surg-ENZ[®]

Regular Foam

For Automatic and Manual/Ultrasonic Use

Advanced MULTI-ENZYMATIC Detergent/Cleaner

Product Features

Rapid enzymatic cleaning action (starts cleaning in less than one minute) - reduces cleaning time by facilitating swift and efficient removal of stubborn organic encrustation.

Fast rinsing and free of metal-attacking agents - rinses quickly and does not contain scale-forming phosphates, chlorides or hydroxides.

Bacteriostatic - ready to use concentrate inhibits bacterial growth.

Economical - highly concentrated; only one fluid ounce (30ml) is required to make one gallon of cleaning solution.

CST-404C multi-enzymatic cleaner is highly effective for fast, efficient removal of dried blood, proteins, fats (lipids), carbohydrates, mucus, albumin, synovial fluids and bone marrow. **CST-404C** is safe and very effective for all flexible and rigid scopes, as well as any instruments composed of stainless steel, aluminum, tungsten carbide, copper, or brass.

Directions for Use

CST-404C is ideally suited for manual and/or ultrasonic scope cleaning, and also as a pre-soak for surgical instruments. Use 1 fl.oz. (30 ml) per gallon of warm water, contingent upon water quality and extent of bio-burden accumulation. **CST-404C** is highly effective at all temperatures, however in autowashers, optimal cleaning and foam control is achieved at 120°-140°F. Allow at least 1-2 minutes cycle time.

Ordering Information

- CST-404C-1** - 4 x 1 gallon bottles packed per case
- CST-404C-5** - 5 gallon plastic jug with handle
- CST-404C-15** - 15 gallon plastic drum
- CST-404C-30** - 30 gallon plastic drum
- CST-404C-55** - 55 gallon plastic drum

Cautions/Warnings

Avoid prolonged contact with skin. May irritate eyes. Wear eye protection as a safety precaution. Do not swallow. **KEEP OUT OF REACH OF CHILDREN!**
Discard used solution - do not reuse.



COMPLETE SOLUTIONS TECHNOLOGIES
CST

Complete Solutions Technologies, LLC
302 N. Washington St., Suite 202 E
Moorestown, NJ 08057

www.cstllc.com
Toll-Free 888-987-1987
Tel 856-234-7392
Fax 856-234-7918