

# Surg-ENZ<sup>®</sup>

## Super Concentrate

### For Instrument and Scope Washers

## ADVANCED MULTI-ENZYMATIC Detergent/Cleaner

### Product Features

**Fast enzymatic cleaning action** (starts cleaning in less than one minute!) - reduces cleaning time by rapid, efficient removal of stubborn organic encrustation.

**Fast rinsing and free of metal-attacking agents** - rinses quickly and does not contain scale-forming phosphates, chlorides or hydroxides.

**Bacteriostatic** - inhibits bacterial growth in a ready-to-use concentrate.

**Economical** - high level of concentration is cost effective.

**CST-404SCLF Multi-Enzymatic Cleaner** is highly effective for fast, efficient removal of dried blood, proteins, fats (lipids), carbohydrates, mucus, albumin, synovial fluids and bone marrow. **CST-404SCLF** is safe and effective for all flexible and rigid scopes, stainless steel, aluminum, tungsten carbide, copper and brass instruments.

### Directions for Use

For use as an automatic washer detergent or as a manual pre-soak for surgical instruments: program machine to deliver 1/8-1 fl. oz. per gallon of water, contingent upon water quality and bio-burden. **CST-404SCLF** is highly effective at all temperatures, but if using an autowasher, optimal cleaning and foam control is achieved at 120°-140° F. Allow at least 1-2 minutes cycle time.

### Ordering Information

**CST-404SCLF-2.5** - 2 x 2.5 gallon bottles packed per case

**CST-404SCLF-5** - 5 gallon plastic jug with handle

**CST-404SCLF-15** - 15 gallon plastic drum

**CST-404SCLF-30** - 30 gallon plastic drum

### Cautions/Warnings

Avoid prolonged contact with skin. May irritate eyes. Wear eye protection as a safety precaution. Do not swallow. **KEEP OUT OF REACH OF CHILDREN!**  
Discard used solution - do not reuse.



COMPLETE SOLUTIONS TECHNOLOGIES  
**CST**

Complete Solutions Technologies, LLC  
302 N. Washington St., Suite 202 E  
Moorestown, NJ 08057

www.cstllc.com  
Toll-Free 888-987-1987  
Tel 856-234-7392  
Fax 856-234-7918