

# Medi-Sheen™

## STAINLESS STEEL RECONDITIONER for MANUAL & MACHINE APPLICATIONS

### Product Features

**Tough on stains, gentle on instruments** - **Medi-Sheen** is a powerful stain and rust remover coupled with corrosion inhibitors to recondition and provide lasting protection to stainless steel instruments.

**Frees up frozen box locks** - in most cases, instruments with frozen box locks that are out of service are restored with **Medi-Sheen**.

**Ideal for cleaning steam autoclaves** - **Medi-Sheen's** robust stain and rust removing agents quickly cut through scale and deposit, typical of steam sterilizers.

**Cost competitive** - **Medi-Sheen** gets the job done faster for less, as compared to similar competitive products.

### Directions for Use

Reconditioning of stainless steel surgical instruments: dilute one part **Medi-Sheen** to seven parts hot water, in a stainless steel tray. Open clean instruments and completely immerse in diluted **Medi-Sheen** solution for 12 minutes (severely stained instruments may require up to 30 minutes). Remove & rinse with water. Treat again by scrubbing with **Medi-Sheen**, then rinse thoroughly with water. Severely stained instruments may require second treatment.

Reconditioning of steam sterilizers & autoclaves: dilute one part **Medi-Sheen** to five parts hot water (120°-170°F). Apply to interior surfaces of autoclave. Allow contact for 30 minutes. Brush & rinse.

Reconditioning of stainless steel trays & carts: dilute one part **Medi-Sheen** to five parts hot water (120°-170°F). Brush stains & rust away with stiff brush & hot solution. Rinse completely following wash procedure.

### Technical Data

pH: 2.0 (1:7 parts volume in deionized water)  
Specific Gravity: 1.15 g/ml

### Order Information

**CST-800-1** - 4 x 1 gallon plastic bottle packed per case  
**CST-800-5** - 5 gallon plastic jug  
**CST-800-15** - 15 gallon plastic drum  
**CST-800-30** - 30 gallon plastic drum  
**CST-800-55** - 55 gallon plastic drum

### Caution/Warnings

**Medi-Sheen** contains phosphoric acid that causes burning of the skin and eyes upon contact. Use of gloves, eye protection and protective clothing is highly recommended. If contact with skin or eyes occurs, wash immediately with copious amounts of water, then call a physician. Refer to MSDS for additional safety information. **Medi-Sheen** should never be mixed with any other product.



COMPLETE SOLUTIONS TECHNOLOGIES  
**CST**

Complete Solutions Technologies, LLC  
302 N. Washington St., Suite 202 E  
Moorestown, NJ 08057

[www.cstllc.com](http://www.cstllc.com)  
Toll-Free 888-987-1987  
Tel 856-234-7392  
Fax 856-234-7918